

### **Change of dispenser material** for **Poseidon Hydrophilic Nitinol Guide Wire** and **Poseidon URO Guide Wire**

From 1<sup>st</sup> of March, Poseidon Hydrophilic Nitinol Guide Wire and Poseidon URO Guide Wire will be packed in **Polyethylene (PE) dispenser** instead of Polypropylene (PP) dispenser.

As PE provides less friction compared to PP,  
it will be easier to move the guide wire inside the dispenser.

The change does not affect the information in the Instructions for Use.  
E.g., Preparations before use must still be followed as it is important to hydrate the guide wire to activate the hydrophilic coating:

#### **Preparation before use:**

- 1) Before removing the guide wire from the dispenser, use a 20 ml syringe to fill the dispenser with saline (sterile) and allow the guide wire to hydrate for at least 30 seconds.
- 2) Carefully remove the guide wire from the dispenser.
- 3) Inspect the guide wire thoroughly to make sure that it is not damaged.

In a transition period, products in PP dispenser may be delivered.

