# EN Specimen Collection Flocked Swab - Instructions for use

## Description:

Specimen Collection Flocked Swab is suitable for collecting and storing human specimens. The device is 150 mm long and  $\varnothing 3$  or  $\varnothing 5$  mm in swab diameter. The swab length is approx. 17 mm. The device is available in different variants of the same flocked swab with various placements of the moulded breakpoints. The device is provided in sterile condition and is for single use only. The device is sterilized by radiation.

Item number	Swab diameter	Breakpoint. Position from swab tip
990000000100	$\varnothing 5$ mm	18 mm
990000000101		31,25 mm & 77,5 mm
990000000102		31,25 mm
990000000103		77,5 mm
990000000104	$\varnothing 3$ mm	77,58 mm

## Intended purpose:

The sterile Specimen Collection Flocked Swab is intended for collection of human specimens.

## Contraindications:

There are no known contraindications.

## Warnings:

1. SINGLE USE. This device is for single use only. Do not re-use. Reuse may cause risk of infection and/or inaccurate results.
2. Do not use if:
  - There is any sign of damage or contamination.
  - The expiry date has passed.
  - The swab package is open.
  - There are other signs of deterioration.

## Precautions:

1. The device should only be selected and used by trained personnel and / or professional users in a controlled environment in hospitals and clinics.
2. Swab shafts have a moulded breakpoint designed for intentional breakage of the swab into a transport tube. Never use excessive force, pressure or bending when collecting swab samples from patients as this may result in accidental breakage of the swab shaft.

## Potential side effects:

Possible complications include, but are not limited to the following:

- Mucous trauma
- Infection

## Serious incident:

Every serious incident that has occurred in relation to the device should be reported without delay to the manufacturer and the competent authority of the Member State in which the user and / or patient is established. For further information please contact the manufacturer.

## Preparations for use:

1. Peel to open the sterile package from the end indicated with an arrow and remove the swab without anything touching the swab tip.
2. Inspect the swab to make certain that it is not damaged.

## Directions for use:

1. Collect specimen and ensure that the swab tip is only in contact with the area from which the sample is to be taken.
2. Process the swab according to internal laboratory instructions. Swab may be placed in a sterile tube and transferred to laboratory for microbiological analysis.

## Storage:

To be stored under dry conditions protected from sunlight. Temperature limits: 4°C –25°C.

## Expiry date:

The device must be used no later than stated on the package.

## Disposal:

Disposal shall be handled in accordance with accepted medical practice and applicable local laws and regulations for hazardous waste.

## Safety information:

The IFU is delivered in hard copy together with product and kept updated and available for electronic download on SP Medical's website.

## Disclaimer of liability:

SP Medical A/S is not liable for defects / deterioration resulting from abnormal use, unauthorized, or unqualified use, or modifications made to the product. Under these circumstances the product is not covered by the guarantee. SP Medical A/S disclaims liability for direct or indirect injuries that may occur as a consequence of the product being modified or wrongly used.

## Explanation of symbols used on the package labels:

	Contains sufficient for <n> tests		Medical Device
	Consult instructions for use, or consult electronic instructions for use		Do not re-use
	Keep away from sunlight		Keep dry
	Temperature limit		Use-by date
	Country of manufacture (DK – Denmark) and date of manufacture		Do not use if package is damaged and consult instructions for use
	Sterilized using irradiation.		Sterilized using irradiation. Single sterile barrier system
	Manufacturer		Sterilized using irradiation. Single sterile barrier system with protective packaging outside
	Reference number		Lot number
	Unique Device Identifier		SP Medical A/S
	CE mark		Møllevvej 1, DK-4653 Karise Phone: +45 56 76 60 00 <a href="http://www.sp-medical.com">www.sp-medical.com</a>

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